


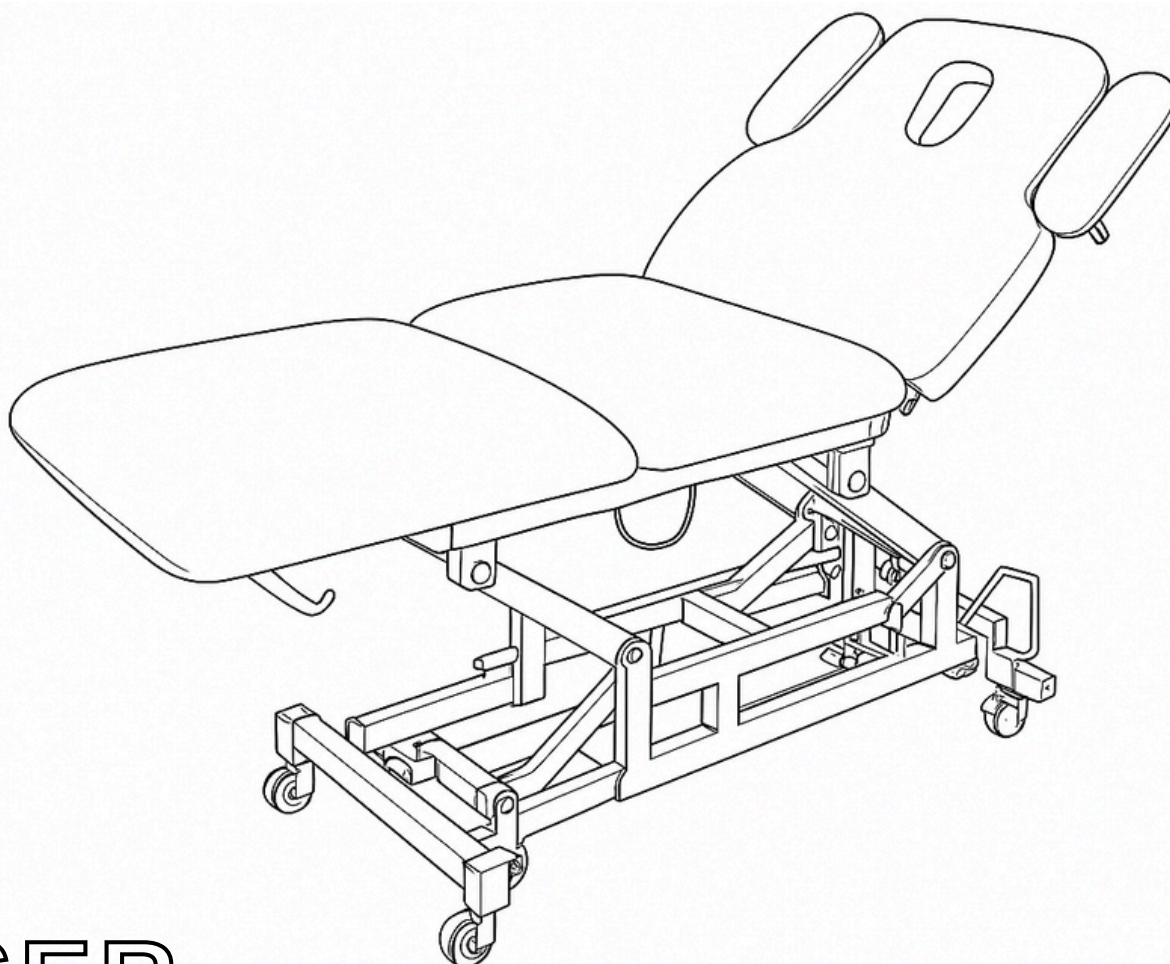

USER MANUAL **ELECTRIC MASSAGE BEDCR-849**

All rights reserved.

Printed in Canada

TABLE OF CONTENTS

1. INTRODUCTION	01
.....	
2. SAFETY GUIDELINES	01
.....	
3. PARTS AND FEATURES	02
.....	
4. INSTALLATION INSTRUCTIONS	03
.....	
5. CLEANING AND MAINTENANCE	03
.....	
6. WARRANTY AND SUPPORT	04
.....	

Please read this manual carefully before using the product to ensure safe and optimal use.

Our company reserves the right to improve design and modify the products without prior notice.

Introduction

Thank you for purchasing the Electric Massage Bed CR-849. This electric massage bed is equipped with three independent motors for professional-grade adjustability. Ideal for spas, beauty clinics, medical offices, and more.

Safety Guidelines

- **Max Weight Capacity:** Always ensure the weight capacity of the bed is not exceeded.
- **Stable Surface:** Always place the bed on a level, stable surface. Do not stand on this equipment.
- **Lock Mechanisms:** Always ensure the height, backrest, and footrest adjustments are locked in place before use. Do not allow children to play with .
- **Do Not Overload:** Avoid excessive force on the adjustable parts of the bed to prevent damage.
- **Electrical Safety:** Always use a grounded electrical outlet, and ensure it is properly connected to the power source.
- This product should be used by an experienced person who must be familiar with the product structure, performance and instructions.
- Tighten bolts and nuts regularly to avoid failure caused by loose, sliding, vibration and falling off.
- This products is not allowed to operate when it's with fault. If abnormal phenomenon or abnormal sound is found, it must be stopped for inspection immediately, and work can be resumed after troubleshooting.
- Maintenance should be processed by trained professionals.
- After every treatment, please only use neutral solution to clear the dust or stains on the product.
- Keep children away from controls.
- For indoor, dry-use only
- Keep hands clear of moving parts

Parts and Features

Features:

- Three-Motor Functions:
 1. Height Adjustment – Raises or lowers the entire bed
 2. Backrest Adjustment – Raises or reclines the backrest
 3. Leg Rest Adjustment – Lifts or lowers the leg section
- Adjustable Prone Armrests:
 1. Located below the face hole/headrest for comfort during face-down treatments
 2. Can be moved forward/backward and up/down depending on client position
- Foam Cushioning: High-density for extra comfort
- PU Leather Upholstery: Easy to clean and durable
- Comfortable headrest with a cut-out for facial treatments
- Easy to move with wheels

Specifications

- Dimensions: 80" x 29" x 25"-38"
- Weight: 190lb
- Material: Steel frame with high-quality faux leather upholstery
- Weight Capacity: Up to 350lb
- Power Requirements: 100–240V, 50/60Hz
- Color Options: Black/ White
- Environment Temperature: 0~40 degree

Installation Instructions

1. Place the bed on a flat, stable surface
2. Plug the power cord into a grounded outlet
3. Use the remote control to adjust bed position

Cleaning and Maintenance

- **Surface: Disconnect power before cleaning or servicing.** Wipe with a damp cloth and mild soap. Avoid strong chemicals or alcohol-based cleaners.
- **Motor:** Keep dry; do not spray liquids near electrical components.
- **Check Connections monthly to ensure safety and stability.**
- **Lubricate joints as needed with silicone spray (not on motor parts).**

General Care

- Keep the bed in a clean, dry area to prevent rusting or damage.
- Please stay the upholstery flat to 180 degree to avoid folding and wrinkles.

Troubleshooting

Issue	Possible	Cause Solution
Bed doesn't move	Power not connected	Check outlet and power cord
Remote not working	Loose connection	Reconnect remote plug
Motor noise or slow	Overloaded or obstructed	Remove excess weight, check for blockage
Armrest unstable	Not locked after adjustment	Reposition and secure firmly

If problem is still not solved after above operations,
please contact Please contact distributor or manufacturer.

Warranty and Support

- **Warranty:** This product comes with a year warranty that covers defects in materials and parts. Failure to follow recommended care will void product warranty. Consumer misuse not covered. Inspection Required. Normal wear and tear not covered. Satisfaction Does Not Apply.
- **Customer Support:** For any issues with assembly, use, or maintenance, please contact our customer service team at **+1 604 438 3399** Monday–Friday, 9:00 AM – 5:00 PM (Local Time) or email to **info@protabeauty.com**
- **Service:** If you need repairs or parts replacement, contact us for assistance.



Do not dispose of old or defective appliances in domestic garbage; this should only be done through public collection points.

▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.