


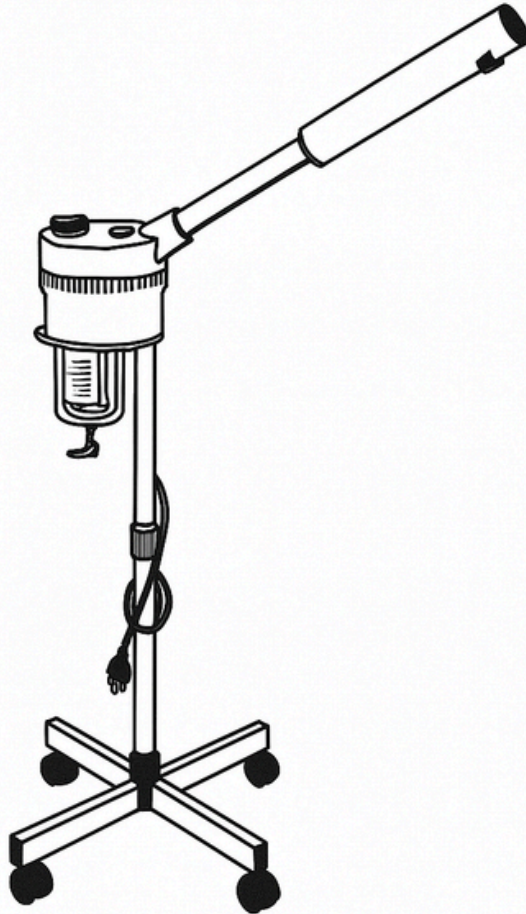

USER
MANUAL
FACIAL STEAMER
PB-1000B

All rights reserved.

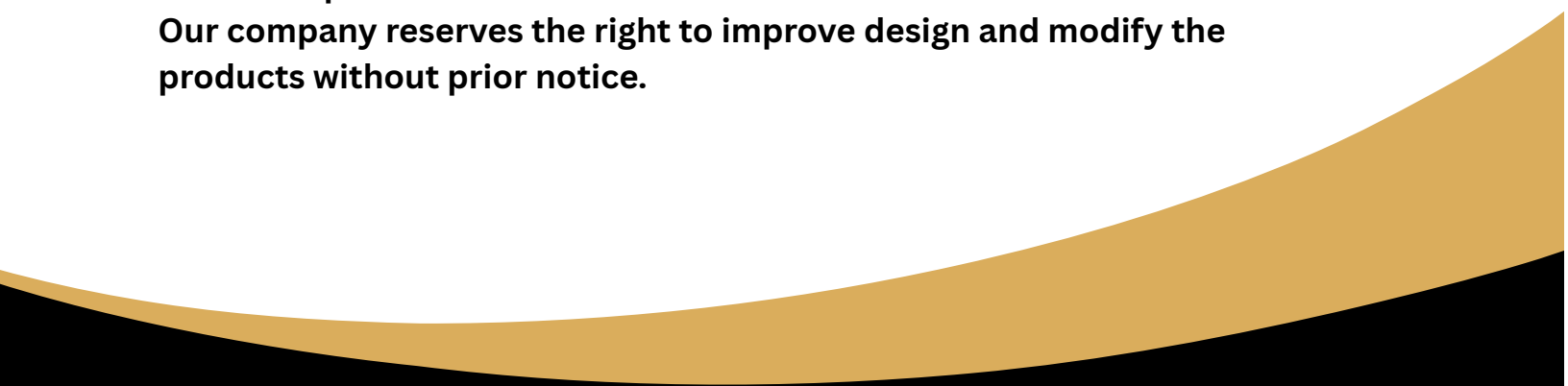
Printed in Canada

TABLE OF CONTENTS

1. INTRODUCTION	01
2. SAFETY PRECAUTIONS	01
3. PARTS AND FEATURES	02
4. OPERATION DIRECTIONS	03
5. CLEANING AND MAINTENANCE	03
6. TROUBLE SHOOTING	04
7. WARRANTY AND SUPPORT	04

Please read this manual carefully before using the product to ensure safe and optimal use.

Our company reserves the right to improve design and modify the products without prior notice.

A decorative wavy line in a golden-brown color, starting from the left edge and curving upwards towards the right, ending at the bottom right corner of the page.

Introduction

The CR-1000B Facial Steamer is a professional-grade steam machine designed for use in beauty salons, spas, and skincare clinics. It emits a fine mist of warm steam that opens pores, hydrates the skin, and enhances the effectiveness of facial treatments. With its adjustable arm, ozone function (if applicable), and mobile base, the CR-1000B delivers convenience and reliable performance for estheticians and skincare professionals.

Safety Precautions

- Use distilled water only to prevent mineral buildup.
- Do not overfill the water tank – always fill to the marked MAX line.
- Always turn off and unplug the unit before filling or cleaning.
- Allow the steamer to cool down completely before touching the water tank or heating element.
- Avoid directing steam into the eyes or towards sensitive areas.
- Keep out of reach of children and pets.
- Do not immerse any part of the steamer in water.
- Ensure the machine is placed on a flat, stable surface.
- Do not operate with a damaged cord or plug.
- Use in a well-ventilated area.

Failure to comply with these instructions will result in the voiding of the manufacturer's warranty.


Parts and Features

Features

- Steam Outlet Nozzle – Adjustable arm directs steam flow
- Control Panel – On/off switches and indicator lights
- Rotating Arm – Adjustable for different angles and client positions
- Clamp Ring – Secures the jar to the heater base
- Water Jar (Glass) – Holds distilled water for steaming
- Heating Element – Located inside the water jar
- Water Level Indicator – Shows minimum and maximum levels
- Safety Clamp – Secures water jar in place
- Telescopic Pole – Adjustable height for user comfort
- Rolling Base – Stable and easy to move
- Power Cord – Standard plug for AC power
- Stimulates Blood Circulation
- Keeping the skin white and tender
- Activating genes and renewing water in skin
- Regenerating protein, making skin tender and smooth
- Softening cutin layer, making skin absorb much nutrition
- Killing bacteria in the skin
- Auto power-off feature

Specifications

- Dimensions: 27.5" x 27.5" Base x 52.5" Height
- Weight: 30lb
- Power Supply: AC 110–240V, 50/60Hz
- Heating Method: Internal coil heater
- Steam Generation Time: Approx. 5–8 minutes
- Steam Duration: runs for 25 minutes per session with a 5-minute reset feature
- Water Jar Capacity: 650ml
- Arm Type: Adjustable, rotating steam arm 37"-42"
- Base Type: 4-leg rolling base with caster wheels
- Nozzle Distance to Face: Recommended 12–16 inches
- Body Material: Heat-resistant ABS and stainless steel
- Glass Jar Material: High-strength borosilicate or tempered glass

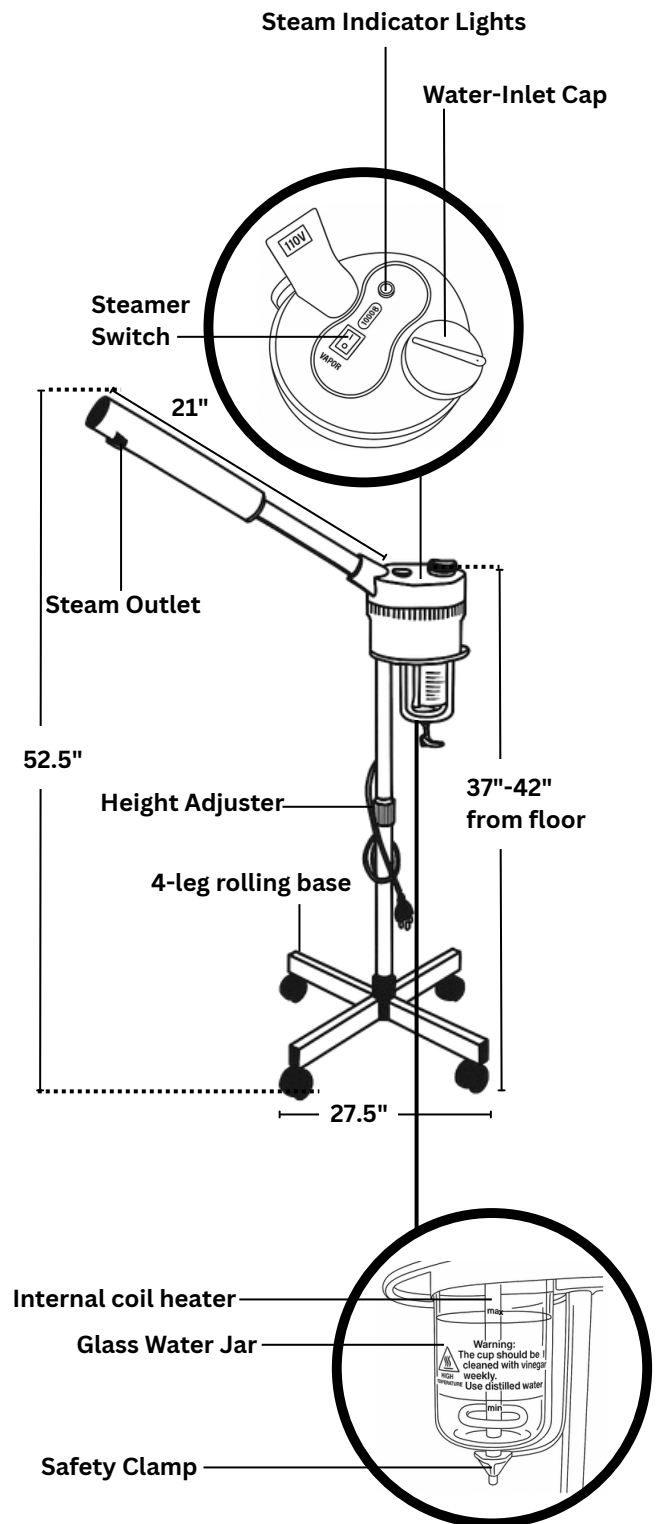

Operation Directions

1. Setup

- Position the unit on a flat, stable surface.
- Adjust the height and arm as needed.

2. Filling the Water Jar

- Unplug the unit.
- Detach the jar and fill with distilled water only up to the MAX line.
- Reattach and secure using the clamp ring.

3. Power On

- Plug in the unit.
- Press the power and steam switches.
- Wait 5–8 minutes for steam to begin emitting.

4. Use

- Position the nozzle 12–16 inches from the face.
- Steam for 10–15 minutes based on skin type and treatment.

5. Power Off

- Turn off all switches.
- Unplug the unit.
- Allow to cool before cleaning or refilling.

Cleaning and Maintenance

After Each Use:

- Allow unit to cool.
- Empty unused water from the jar.
- Wipe nozzle and housing with a soft cloth.

Weekly:

- Soak the water jar in a 1:1 white vinegar and water solution for 15 minutes.
- Rinse thoroughly and dry before next use.

Do not:

- Use tap or mineral water.
- Scrub with abrasive cleaners.
- Immerse the base or electrical parts in water.

Trouble Shooting

Problem	Possible Cause	Solution
No steam	No water / not enough heat time	Fill water / wait 5–8 minutes
Weak or intermittent steam	Mineral buildup on heating element	Clean jar and heating coil
Water leaking	Jar not sealed properly / overfilled	Reseal and refill correctly
Strange odor	Dirty jar or coil	Clean thoroughly with vinegar solution
No power	Plug or switch issue	Check outlet, cord, and power button

Warranty and Support

- **Warranty:** This product comes with a year warranty that covers defects in materials and parts. Failure to follow recommended care will void product warranty. Consumer misuse not covered. Inspection Required. Normal wear and tear not covered. Satisfaction Does Not Apply.
- **Customer Support:** For any issues with assembly, use, or maintenance, please contact our customer service team at **+1 604 438 3399** Monday–Friday, 9:00 AM – 5:00 PM (Local Time) or email to **info@protabeauty.com**
- **Service:** If you need repairs or parts replacement, contact us for assistance.



Do not dispose of old or defective appliances in domestic garbage; this should only be done through public collection points.

▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.