



**USER
MANUAL
ELECTRIC
MEDICAL CHAIR
PB-2168**

TABLE OF CONTENTS

1. INTRODUCTION	01
.....	
2. SAFETY GUIDELINES	01-02
.....	
3. INSTALLATION INSTRUCTIONS	03
.....	
4. PARTS AND FEATURES	04-11
.....	
5. TROUBLE SHOOTING	12
.....	
6. CLEANING AND MAINTENANCE	13
.....	
7. WARRANTY AND SUPPORT	14
.....	

Please read this manual carefully before using the product to ensure safe and optimal use.

Our company reserves the right to improve design and modify the products without prior notice.

Introduction


The Electric Medical Chair is designed to provide a comfortable, adjustable seating position for patients during medical treatments. With multiple adjustable features, the chair helps improve patient comfort and facilitates easy access for medical professionals. Its electric mechanism allows for effortless height, backrest, and leg adjustments.

Safety Guidelines

- Max Weight Capacity: Always ensure the weight capacity of the bed is not exceeded. **Overloading may cause serious injury.**
- Stable Surface: Always place the bed on a level, stable surface.
- Do not stand on this equipment.
- This product must not be used for single-side or single-point loading tasks.
- This product is equipped with a three-wire power cord. It must be connected to a properly installed and grounded three-hole outlet. This prevents electric shock and ensures proper operation.
- Power must be cut off when changing electrical components or maintaining and cleaning machines.
- Do not place the equipment in wet places such as bathroom to avoid electric shock or other failure and keep the controllers away from water. Do not allow children to play with .
- Do Not Overload: Avoid excessive force on the adjustable parts of the bed to prevent damage.
- Electrical Safety: If the bed includes any electrical components, ensure it is properly connected to the power source. Inspect cables for wear, fraying, or deterioration.
- Ensure the plastic structure has no cracks or damage.
- This product should be used by an experienced person who must be familiar with the product structure, performance and instructions.
- This products is not allowed to operate when it's with fault. If abnormal phenomenon or abnormal sound is found, it must be stopped for inspection immediately, and work can be resumed after troubleshooting.

Safety Guidelines

- Always follow the warning labels affixed to the product
- Maintenance should be processed by trained professionals.
- After every treatment, please only use neutral solution to clear the dust or stains on the product.
- When adjusting the lifting mechanism, press and hold the button continuously until the product reaches the desired position.
- When adjusting each part, please keep your hands, feet and head away from the position.

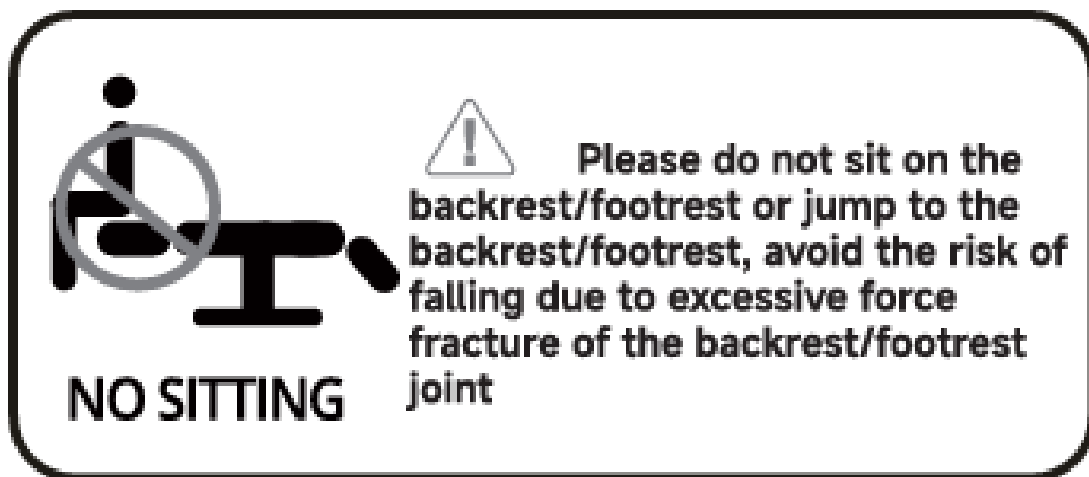

Installation Instructions

1. Unboxing

- Open the box and remove all packing materials.
- The main structure is pre-assembled (excluding headrest, extended footrest, and armrests).
- Carefully take out the bed and all included parts.

2. Assembly Steps


- Insert the two pins from the headrest and extended footrest into the corresponding holes on the cushions.
- Attach the armrests by inserting their pins into the holes on each side of the bed frame (left and right).
- Plug in the power cable.

3. Final Check

- Ensure all components are securely attached.
- Confirm the bed is placed on a stable, level surface.

4. Ready to Use

- Once assembled and plugged in, the bed is ready for immediate use.

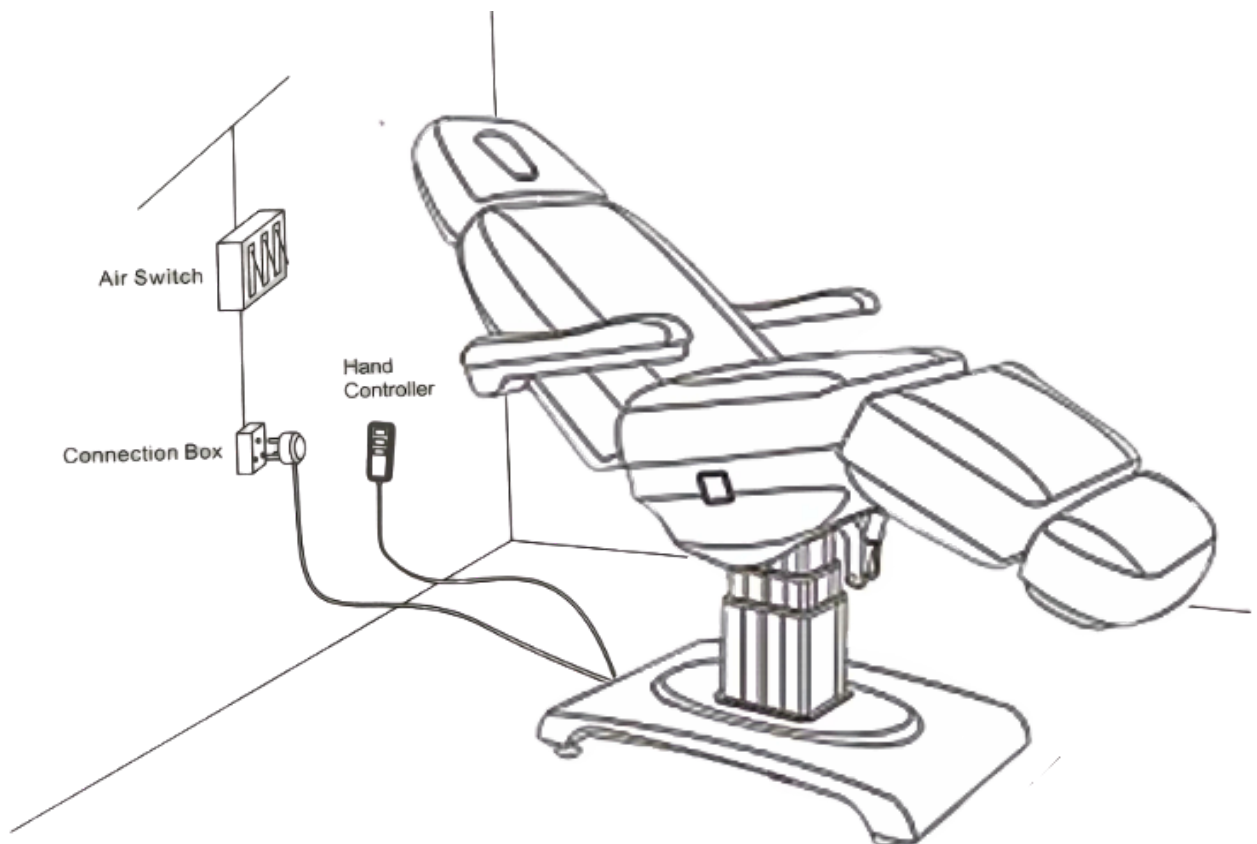

Parts and Features

Features:

- Electric height adjustment for ease of use
- Adjustable backrest and leg rest for patient comfort
- Durable, easy-to-clean upholstery
- Remote control for precise positioning
- Sturdy frame with a weight capacity for medical use
- Smooth and quiet operation for optimal patient comfort
- A face plug is included to seal the face hole when not in use, maintaining a clean and seamless surface.
- A face cushion is included to provide gentle, ergonomic support for clients lying face down during back massages.

Structure of Products


- This product consist of control box, motor, wooden part, hardware, plastic parts and leather. The control box is controlling the operation of the motors and hand controller.
- This product includes angle adjustment of the backrest, angle adjustment of the seat cushion and lifting adjustment which is an overall and integral structure.

Specifications

- Dimensions: 75" x 32" x 26"-35.5"
- Weight: 220lb
- Material: Steel frame with high-quality faux leather upholstery
- Weight Capacity: Up to 400lb
- Power Supply: AC 100-120V or AC 220-240V or AC 100-240V, 50/60Hz
- Maximum Power: 300W
- Color Options: Black/ White
- Environment Temperature: 0~40 degree
- Relative Humidity: 30%~90%

Parts and Features

Foot Controller

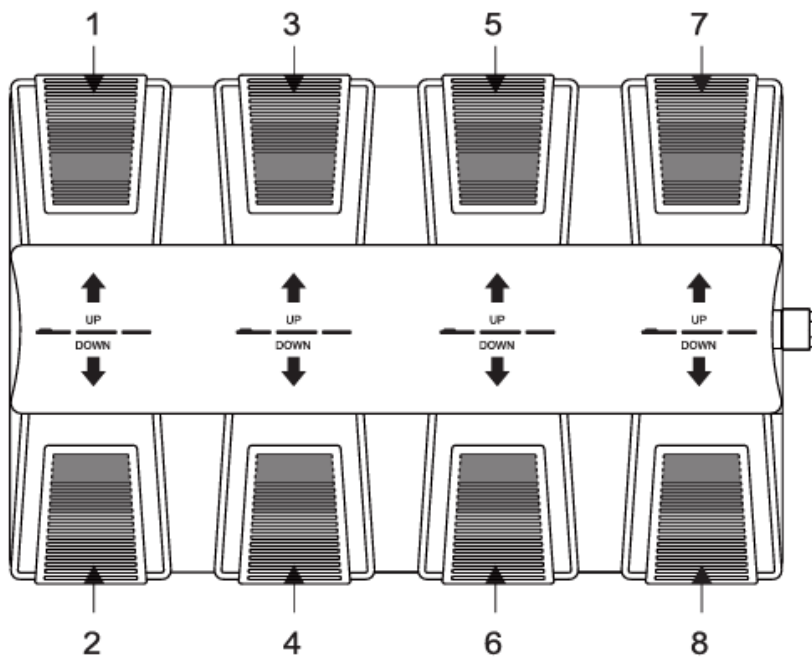

No.	Description	Operation
1	Backrest moves up	Backrest goes up smoothly by pressing the button.
2	Backrest moves down	Backrest goes down smoothly by pressing the button.
3	Whole bed moves up	The bed goes up smoothly by pressing the button.
4	Whole bed moves down	The bed goes down smoothly by pressing the button.
5	Seat cushion up	Seat inclining smoothly by pressing the button.
6	Seat cushion down	Seat goes flat smoothly by pressing the button.
7	Footrest moves up	Angle of right footrest decreases smoothly by pressing the button.
8	Footrest moves down	Angle of right footrest increases smoothly by pressing the button.

Trouble Shooting

The Fault Phenomenon	Reason	Solution
Motor can't work normally	1. The power cord is unconnected or not plugged well.	Connect the power cord or plug it well.
	2. Poor quality of the hand controller or foot controller.	Replace the hand controller or foot controller.
	3. Poor quality of the control box.	Please contact maintenance center or supplier for troubleshooting.
	4. Repture of motor tail.	
	5. Motor was damaged.	
Abnormal Sound	1. The rotating parts are short of lubricating oil.	1. Add lubricating oil.
	2. Rotating positions (such as hinge, POM sets, gas lift) are short of lubricating oil.	2. Add lubricating oil.
	3. Motor quality problems.	3. Replace the motor.

If problem is still not solved after above operations,
please contact Please contact distributor or manufacturer.

Cleaning and Maintenance

Daily Cleaning

- Wipe away daily dirt and grime using a mild soap and warm water solution.
- Dry thoroughly with a soft, lint-free cloth to maintain the surface.

Disinfecting

- Use an isopropanol-based hospital-grade disinfectant approved for polyurethane vinyl.
- Follow manufacturer guidelines to ensure safe and effective disinfection.

Stain Removal

- For minor stains, apply a mild, non-abrasive cleaner to a damp cloth and gently wipe the area.
- Rinse with warm water and dry with a soft cloth.
- Stubborn stains may require professional cleaning.

Warranty and Support

- **Warranty:** This product comes with a year warranty that covers defects in materials and parts. Failure to follow recommended care will void product warranty. Consumer misuse not covered. Inspection Required. Normal wear and tear not covered. Satisfaction Does Not Apply.
- **Customer Support:** For any issues with assembly, use, or maintenance, please contact our customer service team at **+1 604 438 3399** Monday–Friday, 9:00 AM – 5:00 PM (Local Time) or email to **info@protabeauty.com**
- **Service:** If you need repairs or parts replacement, contact us for assistance.



Do not dispose of old or defective appliances in domestic garbage; this should only be done through public collection points.

▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.