


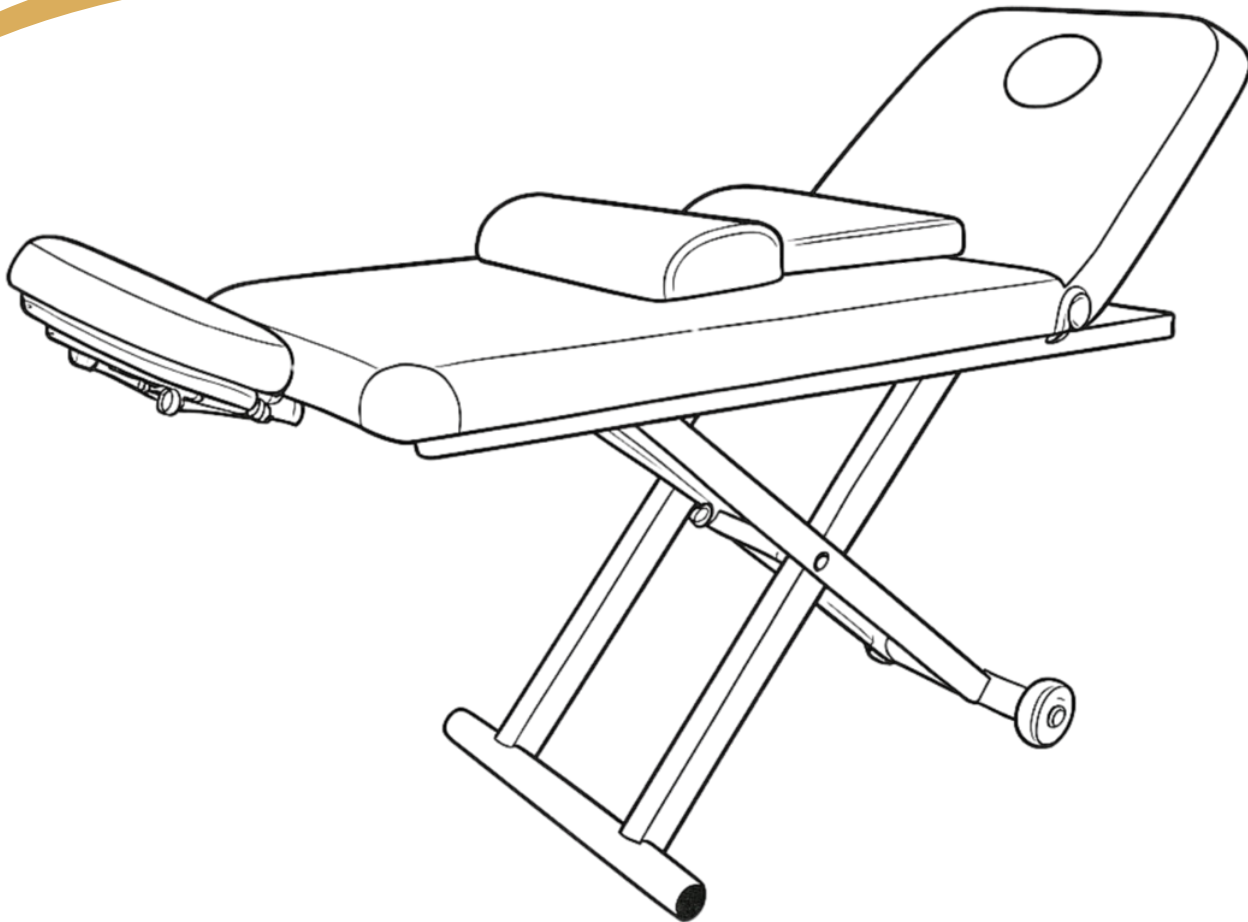

USER MANUAL **ELECTRIC MASSAGE BED PB-830**

All rights reserved.

Printed in Canada

TABLE OF CONTENTS

1. INTRODUCTION	01
.....	
2. SAFETY GUIDELINES	01
.....	
3. PARTS AND FEATURES	02
.....	
4. INSTALLATION INSTRUCTIONS	03-05
.....	
5. CLEANING AND MAINTENANCE	06
.....	
6. WARRANTY AND SUPPORT	07
.....	

Please read this manual carefully before using the product to ensure safe and optimal use.

Our company reserves the right to improve design and modify the products without prior notice.

Introduction

Thank you for purchasing the Electric Massage Bed CR-830. This electric massage bed is equipped with one independent motor for professional-grade adjustability. Ideal for spas, beauty clinics, medical offices, and more.

Safety Guidelines

- **Max Weight Capacity:** Always ensure the weight capacity of the bed is not exceeded.
- **Stable Surface:** Always place the bed on a level, stable surface. Do not stand on this equipment.
- **Lock Mechanisms:** Always ensure the height, backrest, and footrest adjustments are locked in place before use. Do not allow children to play with .
- **Do Not Overload:** Avoid excessive force on the adjustable parts of the bed to prevent damage.
- **Electrical Safety:** Always use a grounded electrical outlet, and ensure it is properly connected to the power source.
- This product should be used by an experienced person who must be familiar with the product structure, performance and instructions.
- Tighten bolts and nuts regularly to avoid failure caused by loose, sliding, vibration and falling off.
- This products is not allowed to operate when it's with fault. If abnormal phenomenon or abnormal sound is found, it must be stopped for inspection immediately, and work can be resumed after troubleshooting.
- Maintenance should be processed by trained professionals.
- After every treatment, please only use neutral solution to clear the dust or stains on the product.
- Keep children away from controls.

Parts and Features

Features:






- One-Motor System: Controls the height of the bed (up/down)
- Adjustable Backrest: Manual mechanism
- Foam Cushioning: High-density for extra comfort
- PU Leather Upholstery: Easy to clean and durable
- Easy to move with wheels
- Comfortable headrest with a cut-out for facial treatments
- Extra inserted headrest (90 degree)

Specifications

- Dimensions:
830 White: 74" x 30.5" x 27"-36"
830 Black: 74" x 30.5" x 27"-36"
- Weight: 150lb
- Material: Steel frame with high-quality faux leather upholstery
- Weight Capacity: Up to 350lb
- Power Requirements: 100–240V, 50/60Hz
- Color Options: Black/ White
- Environment Temperature: 0~40 degree

Installation Instructions

Parts List

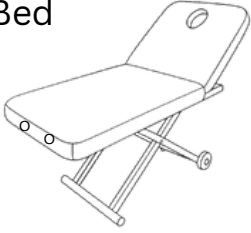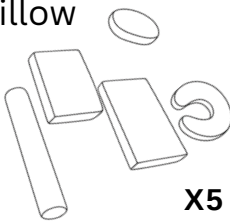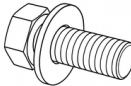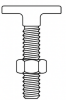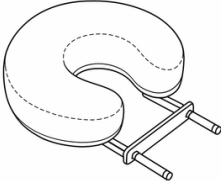
(1) Bed 	(2) Pillow  X5	(3) Base Screw with Washer  X2	(4) Base Screw Cap  X2
(5) Screws with Bolts  X4	(6) Headrest 	(7) Supporting Bars  X2	(8) Screws  X2

Step 1: Unbox and Inspect

Place the product to a proper location and position. Carefully unbox the facial bed and check all components listed in the parts list. Ensure that all parts are in good condition and undamaged. Please contact distributor or manufacturer if there's any problem.

Step 2: Attach the Bed with Supporting Bar


1. Lay the bed on it's side.
2. Attach the metal bars to the bed. Secure them with screws.

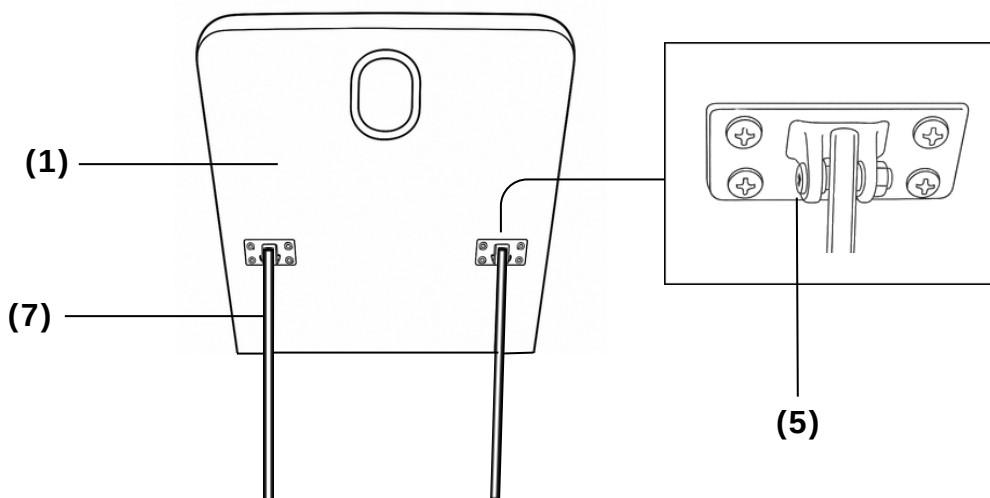

Installation Instructions


Step 3: Secure the Base

1. Align the bed frame with the mounting holes on both sides of the base.
2. Insert screws with washers on both sides.
 - **Be sure to place the washer first, then the screw.**
3. Hand-tighten initially to align, then fully tighten with the provided tool.
4. Cover with the cap.

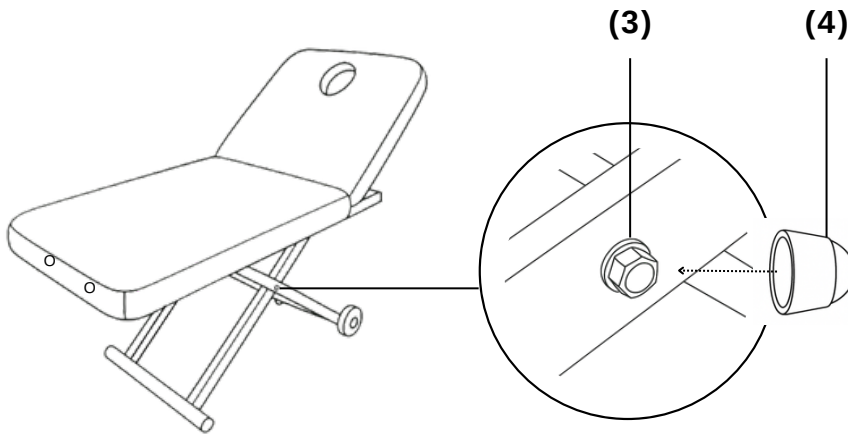

Step 4: Attach the Supporting Bar with Base


1. Insert the previously installed metal bars through the holes of the bar control, then secure with the screw.

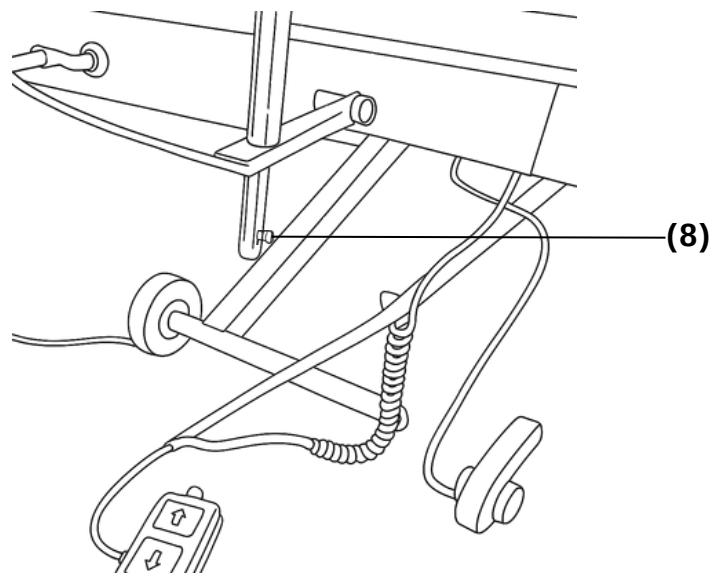

Installation Instructions


Step 5: Attach the Headrest

1. Slide the headrest into the holes at the bottom of the bed.
2. Make sure it is firmly in place. If your model includes locking screws, tighten them for extra security.

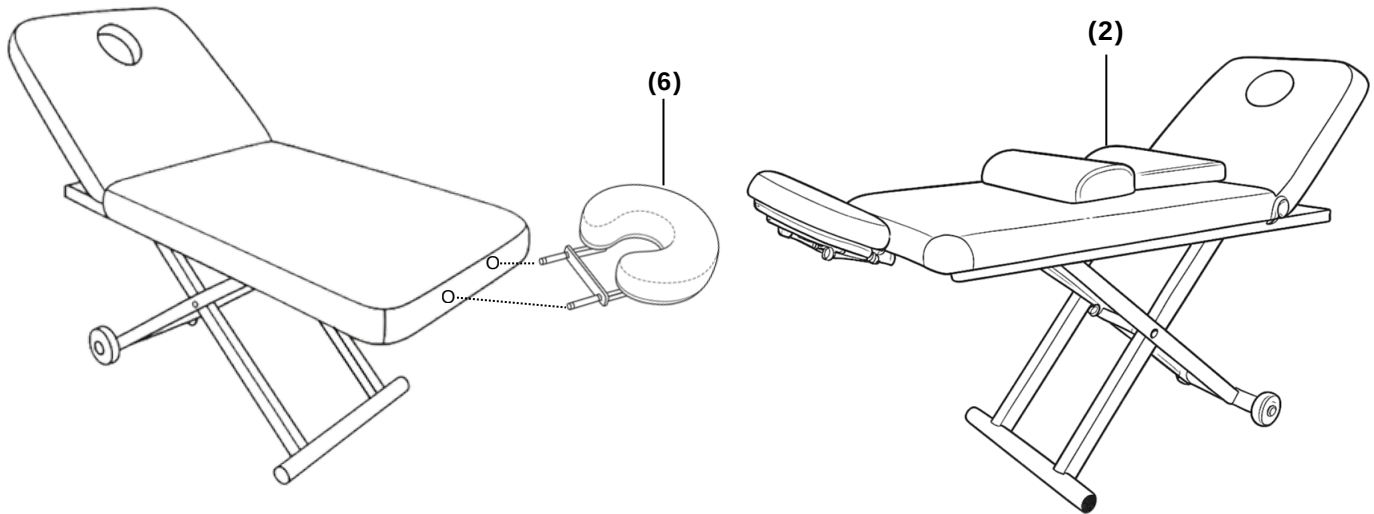

Step 6: Power Connection

1. Plug the main power cords into a grounded outlet.

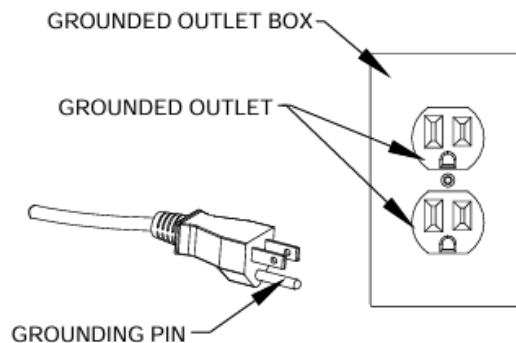

✓ Final Checks

- All screws are securely fastened, and washers are in place.
- Support bar is firmly installed with no movement.
- Headrest is stable and properly attached.
- Power and remote control are connected, and bed raises/lowers smoothly.

Cleaning and Maintenance

- **Surface:** Disconnect power before cleaning or servicing. Wipe with a damp cloth and mild soap. Avoid strong chemicals or alcohol-based cleaners.
- **Motor:** Keep dry; do not spray liquids near electrical components.
- **Check Connections monthly to ensure safety and stability.**
- **Lubricate joints as needed with silicone spray (not on motor parts).**

General Care

- Keep the bed in a clean, dry area to prevent rusting or damage.
- Please stay the upholstery flat to 180 degree to avoid folding and wrinkles.

Troubleshooting		
Issue	Possible	Cause Solution
Bed doesn't move	Power not connected	Check outlet and power cord
Remote not working	Loose connection	Reconnect remote plug
Motor noise or slow	Overloaded or obstructed	Remove excess weight, check for blockage
Backrest not adjustable	Manual mechanism stuck	Check lever or release pin

If problem is still not solved after above operations,
please contact Please contact distributor or manufacturer.

Warranty and Support

- **Warranty:** This product comes with a year warranty that covers defects in materials and parts. Failure to follow recommended care will void product warranty. Consumer misuse not covered. Inspection Required. Normal wear and tear not covered. Satisfaction Does Not Apply.
- **Customer Support:** For any issues with assembly, use, or maintenance, please contact our customer service team at **+1 604 438 3399** Monday–Friday, 9:00 AM – 5:00 PM (Local Time) or email to **info@protabeauty.com**
- **Service:** If you need repairs or parts replacement, contact us for assistance.



Do not dispose of old or defective appliances in domestic garbage; this should only be done through public collection points.

▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.